


# Set up your Medopad Patient Monitoring app

## 1 Activation Code

Before you start with your Medopad Patient Monitoring app, your care provider will provide you with the activation code, without the code you will not be able to access the app.


## 2 Download the Medopad app

Go to the App Store and search for Medopad, Select and make sure you download the 'Medopad Patient monitoring' app.

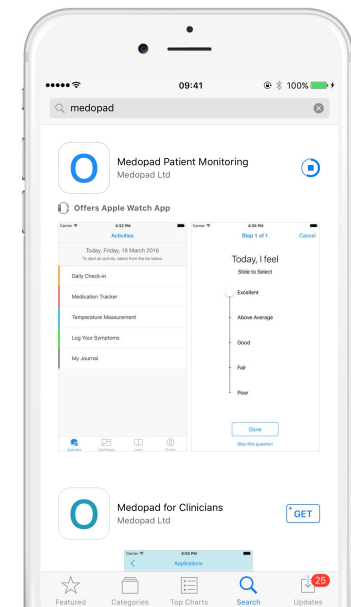


## 3 Register & create an account

Once you have downloaded the app on to your iPhone, select 'register' from the menu at the bottom of your login screen. You can now register by entering your email address, name, password, and registration code.

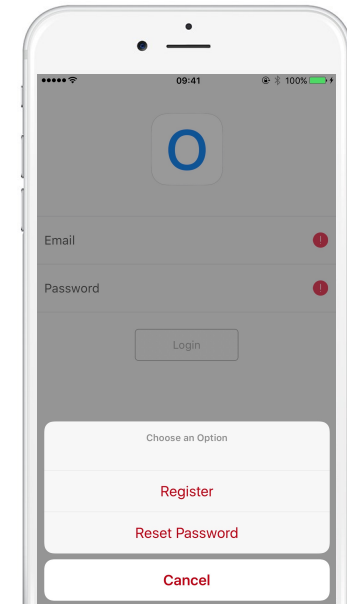

## 4 Activate your account & start using your app


Once you complete your registration, you will receive a confirmation email asking you to activate your account. To start using your Medopad Patient Monitoring app, please check your email and confirm to activate your app.

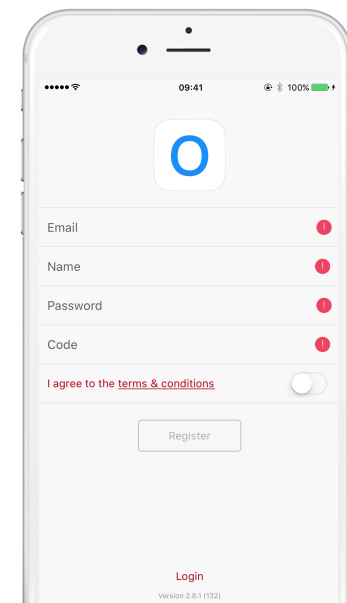

# 5

## Data Access

When you log in for the first time after confirming your activation via email, the data access options page will pop up. Please select 'Turn All Categories On', all the sliders below should go green. This will enable the Medopad to use your phone's inbuilt sensors to collect data such as steps.

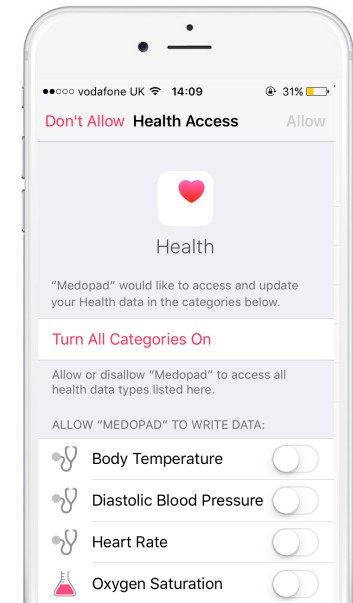

Thank you for activating your Medopad Patient Monitoring app.

We look forward to supporting you during your treatment.

Wishing you the best of health,

The Medopad team